

ARJOHUNTLEIGH

GETINGE GROUP

NEPTUNE

MANUALLY OPERATED
POOL LIFT



...with people in mind



TRANSFER AND LIFTING SYSTEM FOR SITTING PATIENTS

The ArjoHuntleigh Pool Lift gives your patient a safe, comfortable and dignified way of transferring from the poolside into the water. It is also easy and effortless for you, the operator, to use.

The system includes chairs and a comfortable stretcher which attaches easily to the lift. The ArjoHuntleigh Pool Lift suits above ground pools as well as deck level pools.



The ArjoHuntleigh Pool Lift chair transports the patient and attaches to the lift. The fold-back armrests give additional security. After the chair is attached to the lift, remove the sub-chassis. The swivelling castors will help you to do this smoothly.



Fold back one armrest. The patient can now ease themselves from the chair with some assistance.



The ArjoHuntleigh Pool Lift mast can be locked in any desired position. This is useful, for example, when you wish your patient to stay on the chair for hydrotherapy exercises.

TRANSFER AND LIFTING SYSTEM FOR RECUMBENT PATIENTS



The chair may be used in the changing room, in the shower and for toileting.



Using more than one chair makes the transportation and transfer of several patients to the pool quick and efficient.



The ArjoHuntleigh Pool Lift swim-chair has a permanently attached subchassis which makes it possible for your patient to remain seated on the chair for exercises.



The stretcher is wide and comfortable, and the trolley has swivelling casters which make it easy to operate.



Attach the stretcher to the lift. Detach the trolley. Lower your patient into the water. This is a smooth, safe operation.



Your patient can stay on the stretcher for their exercises, or, with some assistance, ease themselves into the water.

ACCESSORIES FOR THE CHAIR, SWIM-CHAIR AND LIFT



A leg rest for the ArjoHuntleigh Pool Lift chair makes transfers into above ground pools smoother.



The Pool Lift can be secured to the floor either by means of a socket or by a base plate.



The ArjoHuntleigh Pool Lift chair has armrests which fold back. The removable chassis is equipped with brakes on all four casters to make the transfer to and from it safe and simple. All parts are made of plastic or stainless steel.



The ArjoHuntleigh Pool Lift swimchair also has folding armrests. The fixed chassis is made of high quality materials and specially designed for use in water. The large front wheels make it easy to move in the pool and the brakes on the rear casters make all transfers safe.




The ArjoHuntleigh Pool Lift stretcher has a comfortable mattress. Velcro makes it easy to remove for cleaning etc. The ArjoHuntleigh Pool Lift stretcher trolley is made of stainless steel.



The ArjoHuntleigh Pool Lift transporter makes lifting and moving the Pool Lift to a store, or from one installation point to another in the pool area, virtually effortless.

PRODUCT SPECIFICATIONS

Space Requirement

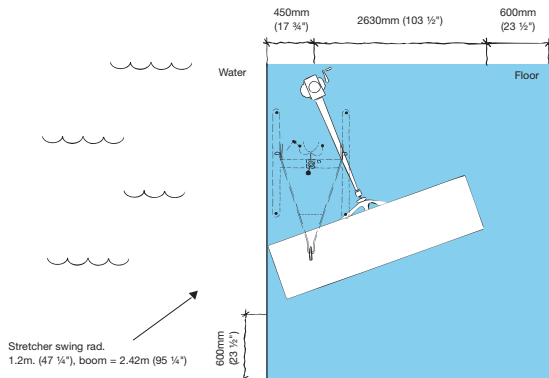

Blue area shows the minimum working area required for the staff to be able to use the mechanical aids in an ergonomic way from one side.

Product Information

Safe working load	160kg (352lbs)
Lifting stroke	1270mm (50")
Lifting stroke above water – min	680mm (26 3/4")
Lifting stroke above water – max	590mm (23 1/2")
Total product height	2340mm (92")

Accessories

- Leg Rest for the Pool Lift Chair
- Large wheels on the chair. The wheels can be used on both chairs, include fastening arrangements
- Pool Lift Transporter

Only ArjoHuntleigh designed parts, which are designed specifically for the purpose, should be used on the equipment and products supplied by ArjoHuntleigh. As our policy is one of continuous development we reserve the right to modify designs and specifications without prior notice.




© and ™ are trademarks belonging to the ArjoHuntleigh group of companies.
© ArjoHuntleigh, 2009.

Neptune Pool Lifts are manufactured on behalf of ArjoHuntleigh by:
Nibotechnics N.V.
Helkant 5
B-3930 Hamont-Achel
Belgium



Mobility Gallery

Suitable for Albert, Barbara, Carl and Doris.

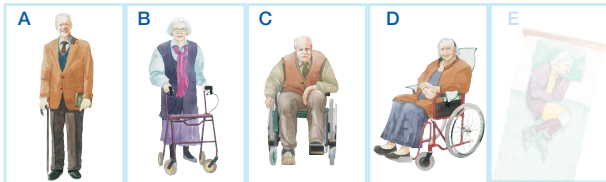

- A Albert, who is ambulatory and independent.
- B Barbara, who can support herself to some degree.
- C Carl, who sits in a wheelchair and has little capacity to support himself.
- D Doris, who has no capacity to support herself.

Please contact ArjoHuntleigh for further information on the Mobility Gallery™.

ARJOHUNTLEIGH

GETINGE GROUP

www.ArjoHuntleigh.co.uk

ArjoHuntleigh UK – Head Office

ArjoHuntleigh House,
Houghton Hall Business Park,
Houghton Regis, Bedfordshire,
LU5 5XF, United Kingdom
Sales Phone: +44 (0) 1582 745700
Sales Fax: +44 (0) 1582 745745
Rental 24hr Helpline
Lo-call Phone: +44 (0) 8457 342000
Rental Fax: +44 (0) 1582 745843
Email: sales.admin@ArjoHuntleigh.com

ArjoHuntleigh UK

Service and Technical Centre
St Catherine Street,
Gloucester, GL1 2SL,
United Kingdom
Phone: +44 (0) 8456 114114
Service Fax: +44 (0) 1452 525207
Email: ukservice@ArjoHuntleigh.com

ArjoHuntleigh (Ireland)

EA House,
Damastown Industrial Park,
Mulhuddart, Dublin 15,
Republic of Ireland
Phone: 01 809 8960
Fax: 01 809 8971
Email: dublin@ArjoHuntleigh.ie
www.ArjoHuntleigh.ie

ArjoHuntleigh (Ireland)

19 Herson Road,
Sydenham Business Park,
Belfast, BT3 9LE,
Northern Ireland
Phone: 028 9050 2000
Fax: 028 9050 2001
Email: belfast@ArjoHuntleigh.com
www.ArjoHuntleigh.ie

GETINGE GROUP is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of **ArjoHuntleigh**, **GETINGE** and **MAQUET**, **ArjoHuntleigh** focuses on patient mobility and wound management solutions. **GETINGE** provides solutions for infection control within healthcare and contamination prevention within life sciences. **MAQUET** specializes in solutions, therapies and products for surgical interventions, interventional cardiology and intensive care.